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ADVERTENCIA

- Use a soft, damp cloth to clean the surfaces of the unit.
- NOTE**– The flow generator release button must be pressed completely to separate button to separate (Fig. 9).

- Turn off the heater and unplug from the power source. Allow the system to cool for at least 10 minutes.
- To detach the flow generator from the humidifier cradle, first remove the water chamber, then fully depress the flow generator release button to separate (Fig. 9).

NOTE– The humidifier cradle should be cleaned as needed.

Humidifier Cradle

CAUTION– The disassembled chamber may also be washed once per week in a dishwasher on the delicate or glassware cycle, top shelf only.

5. Allow the parts to dry before reassembly.

humidifier components or tubing. These solutions may reduce the life of the product.

CAUTION– Do not use any solutions that contain bleach, chlorine, alcohol, scented oils or other aromatic ingredients to clean the humidifier chamber.

- Remove the sealing gasket from the chamber lid (Fig. 8).
- Separate the chamber halves by pressing forward on the chamber disassembly lever and lifting the lid of the chamber (Fig. 7).

4. Lave les pièces en une solution de agua tibia y detergente para lavavajillas suave.

1. Allow device to cool down for at least 10 minutes. Press down on the chamber release latch and pull out the water chamber from the humidifier Water Chamber

CLEANING INSTRUCTIONS

- Clean the chamber according to the cleaning instructions and reassemble.

- Open the chamber and empty any water remaining from the previous night.

- Press down on the chamber release latch of the humidifier and remove the chamber from the cradle (Fig. 2).

humidifier cradle.

NOTE– Always allow at least 10 minutes of cooling time after turning the device off before removing the water chamber from the humidifier cradle.

Daily Use Instructions

The heater setting of 1 results in a heater plate temperature of approximately 149°F (65°C). The heater setting of 10 results in a heater plate temperature of approximately 149°F (65°C).

necessary for optimum comfort.

The heater setting can be adjusted in a range of 1 to 10. The optimum temperature setting is dependent on the ambient temperature and humidity of your room. If your room temperature is low and/or the relative humidity is high, then a lower temperature setting may be necessary for optimum comfort.

Controlling the Humidity

- Heater Power LED

1. Heater Setting Control Buttons

The heater controls and status are located on the keypad and display.

Heater Controls (Fig. 6)

OPERATING INSTRUCTIONS

Do not reach the heater if the water chamber is empty. The heater plate may be turned off using the keypad heater control when being used without water.

Do not reach temperatures as high as 149°F (65°C) during operation.

Never touch the heater plate on the humidifier cradle. Never touch the heat transfer plate on the bottom of the water chamber. These plates can reach temperatures as high as 149°F (65°C) during operation.

If the system is placed on the floor, do not position it where it may be stepped on when getting out of bed.

Do not attempt to fill the humidifier while it is attached to the humidifier cradle. Damage to the humidifier cradle may occur.

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EN	WARNING
<ul style="list-style-type: none">Do not attempt to fill the humidifier while it is sitting in the water tank. If you place the system on the floor, do not place it on a spot where you can trip over it when you get up. Do not touch the heating plate on the humidifier housing or the heat transfer plate on the water tank. These plates can become very hot during use. Do not use the heater if the water tank is empty. The heater can be switched off from the touch panel when the device is used without water.	

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